



Assistive Listening System



FOR USE IN
FULLERTON HALL ONLY

DIRECTIONS

- Turn the headset on by pressing the power button.
- There's no need to select a channel—the device is preset.
- Adjust the volume using the dial at the top of the device.
- During use, position the device with the logo facing outward and the controls at the bottom.
- For best results, do not cover the device with hands or clothing.
- Have a seat anywhere—devices work throughout the auditorium.
- When the program concludes, please return the device to a staff member at the auditorium from which you picked it up.

Thank you!