



Assistive Listening System



DIRECTIONS

- No need to power on—the receiver turns on automatically.
- Select your preferred attachment:
 - T-coil loop:** Set your own hearing aid to “t-coil,” plug the loop into each of the jacks at the top corners of the receiver, and place the loop around your neck.
 - Single-ear headphone:** Plug the headphone jack plug into one of the jacks at the top corners of the receiver and place the headphone over your ear.
 - Personal headphones:** Plug any set with a 1/8" jack plug into one of the jacks at the top corners of the receiver.
- Using the buttons on the front of the receiver, begin with low volume (about 20%) and adjust to your preferred level.
- During use, position the device with volume buttons facing outward.
- For best results, do not cover the device with hands or clothing.
- Have a seat anywhere—devices work throughout the auditorium.
- When the program concludes, please return the device to a staff member at the auditorium from which you picked it up.

Thank you!